

# The Only One-Step On-Site Oral Fluid Device

## Oratect® II Oral Fluid Drug Screen Device


### Drugs of Abuse Test Cut-off Levels\*:

Methamphetamines** (ME)	25 ng/mL
Marijuana*** (TH)	40 ng/mL
Cocaine (CO)	20 ng/mL
Amphetamines (AM)	25 ng/mL
Opiates (OP)	10 ng/mL
Phencyclidine (PC) or	4 ng/mL
Benzodiazepines (BZ)	5 ng/mL

\*Concentration in undiluted neat oral fluid

\*\*Includes Ecstasy (MDMA) at 25 ng/mL

\*\*\*Parent THC

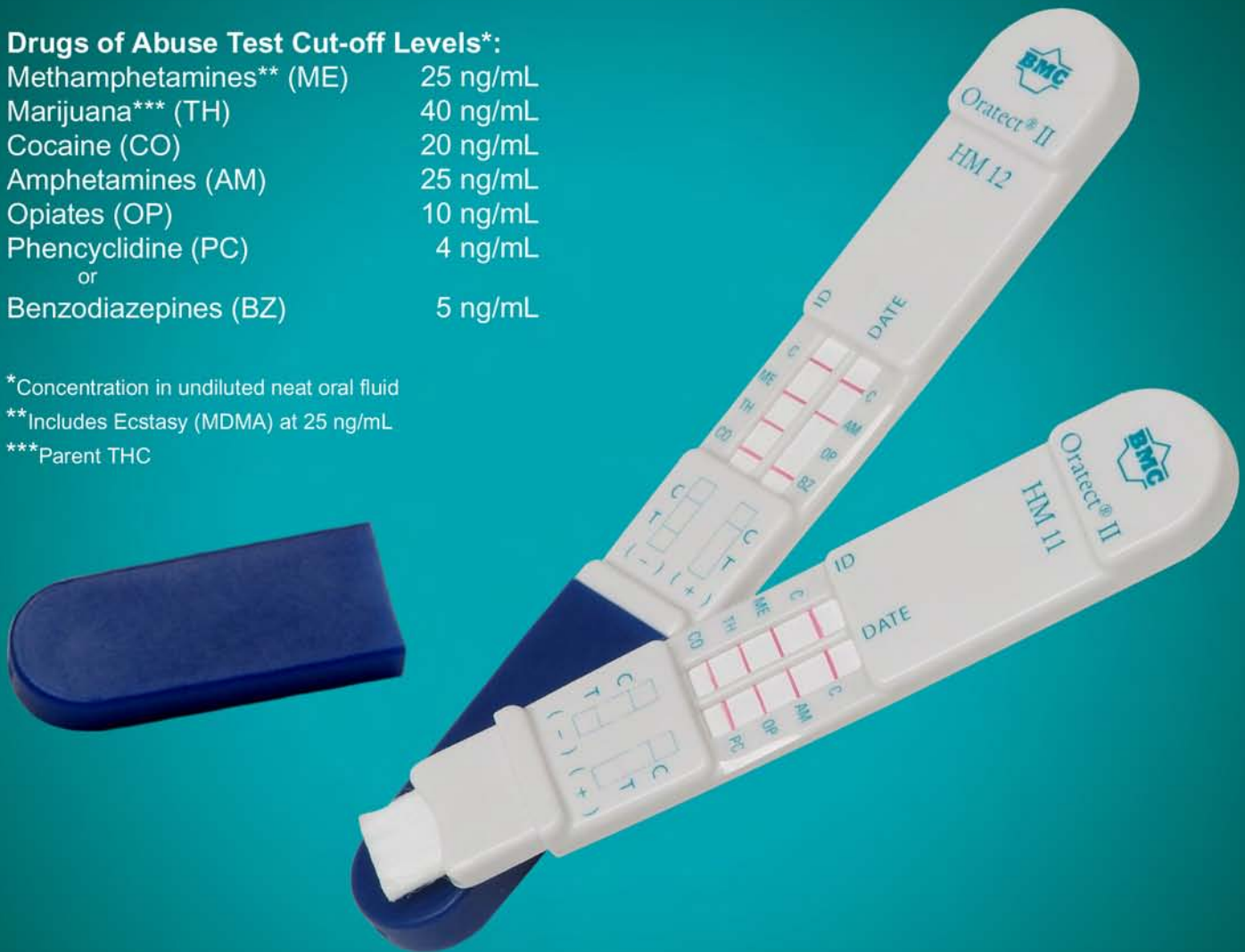

- Collection and testing in one integrated device
- Increased sensitivity of drug cut-off levels
- Two test configurations of 6 drugs
- Donor performed test with minimal administrator supervision
- Rapid collection and testing; results in 5 min
- User-friendly and non-invasive
- Avoids gender observation
- Eliminates sample adulteration or contamination
- Built-in confirmation for chain-of-custody testing





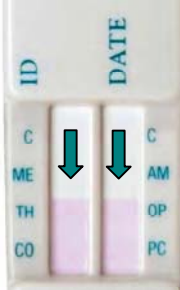





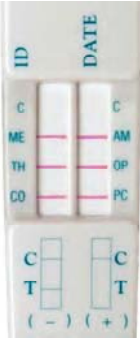


# Oratect® II Reference Guide

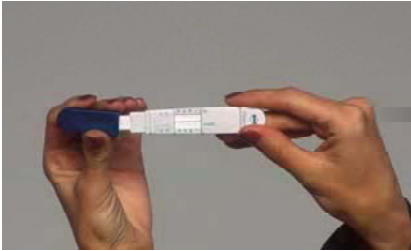The Oratect® II\* Oral Fluid Drug Screen Device is a simple one-step test for the detection of drugs of abuse in oral fluid.

**Oratect® II Procedures** -- Refer to package insert for detailed instructions and technical information.

<p><b>1. Remove</b> the blue cap by holding the sides and pulling gently. This will expose the collection pad. <b>Do not touch the test window area.</b></p> 	<p><b>2. Open</b> mouth and <b>rub</b> the collection pad inside mouth against cheek in a circular motion twenty times. Make sure to <b>keep head level.</b></p> 	<p><b>3. Rub</b> the collection pad against the opposite cheek in circular motion twenty times.</p> 
<p><b>4. Rub</b> the collection pad <b>on top</b> of the tongue twenty times. <b>Do not chew, suck, bite or bend the collection pad.</b></p>  <p><b>5. Rub</b> the collection pad <b>underneath</b> the tongue twenty times.</p>	<p><b>6. Place</b> the collection pad underneath the tongue to collect saliva. Instruct the donor to hold the device in place with hand. <b>If no pink-purple flow appears (indicated by arrows) in the test windows after 30 seconds, repeat steps 2-6.</b></p>  <p style="text-align: center;"><b>Test Windows</b></p>	<p><b>7. Remove</b> from mouth as soon as pink-purple flow patterns appear at both of the test windows. <b>Cap</b> the device with the blue cap.</p>  <p><b>8. Lay</b> the device on a flat surface and <b>read results in 5 minutes</b> after removing the device from mouth. <b>Do not read results after 7 minutes.</b></p>

## Interpreting Oratect® II Test Results

<p style="text-align: center;"><b>Invalid Result</b></p> <p>When <b>no</b> band appears in the <b>CONTROL (C) region</b>, the test is <b>invalid</b> even if there is a band in the test region. <b>Repeat</b> the test with a new device.</p>  <p>Example Interpretation:</p> <p>ME: Invalid TH: Invalid CO: Invalid AM: Invalid OP: Invalid PC: Invalid</p>	<p style="text-align: center;"><b>Negative Result</b></p> <p>For each test, two colored bands should be observed:</p> <ul style="list-style-type: none"> <li>• <b>One in the CONTROL (C) region</b></li> <li>• <b>One in the specific TEST region</b></li> </ul> <p>The color of the test band may be slightly darker or lighter than the control band. Any visible band that can be seen is a <b>negative</b> result.</p>  <p>Example Interpretation:</p> <p>ME: Negative TH: Negative CO: Negative AM: Negative OP: Negative PC: Negative</p>	<p style="text-align: center;"><b>Presumptive Positive Result</b></p> <p>A colored band at the <b>CONTROL (C) region</b> should be observed. When there is no colored band at the specific <b>TEST region</b>, the test is presumptive positive for that particular drug.</p>  <p>Example Interpretation:</p> <p>ME: <b>Positive</b> TH: Negative CO: Negative AM: Negative OP: Negative PC: Negative</p>
--	--	--